


Operating Instructions

1

ASSESS PATIENT CAN WEIGHT BEAR

IF USING THE HEIGHT ADJUSTABLE
ROTUNDA, SET TO THE REQUIRED
HEIGHT BEFORE USE

2

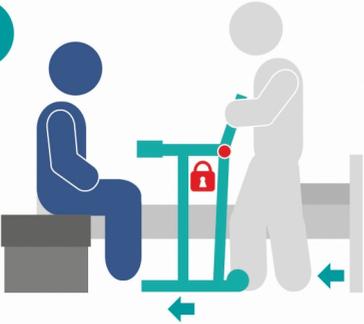

The assistant wheels the Rotunda into position in front of the seated patient and sets it down, then depresses the foot pedal and slides the Rotunda under the patients feet.

3



The patient places their feet onto the turntable and using the handle grip area to hold moves to a standing position (with assistance if needed).

4



The assistant unlocks and rotates the turntable smoothly left or right to engage the brake rod into one of the five locking positions.

5



The assistant can now transfer the patient to a previously placed wheelchair, commode, chair or bed, the assistant then removes the Rotunda by tilting it back on to the wheels and moving it away.

**THE ROTUNDA IS A
TRANSFER AID ONLY AND
MUST NOT BE USED TO
TRANSPORT PATIENTS**

